

Manual/ Specifications

Portable Potty Seat



Round or Elongated

Parts



Soft-Touch™ potty seat

1. Place the portable potty seat over your existing toilet seat.

Pommel toward the front



! **WARNING:** Prevent serious injury or death from falls or sliding out

- Do not submerge
- Never leave child unattended
- Always keep child in view




Bergeron By Design
15 South Second St.
Dolgeville, NY 13329
1.866.529.8407

Model# Series 74XXXXXX
Revision 03/13
UMPPS03-13LIT



Specifications

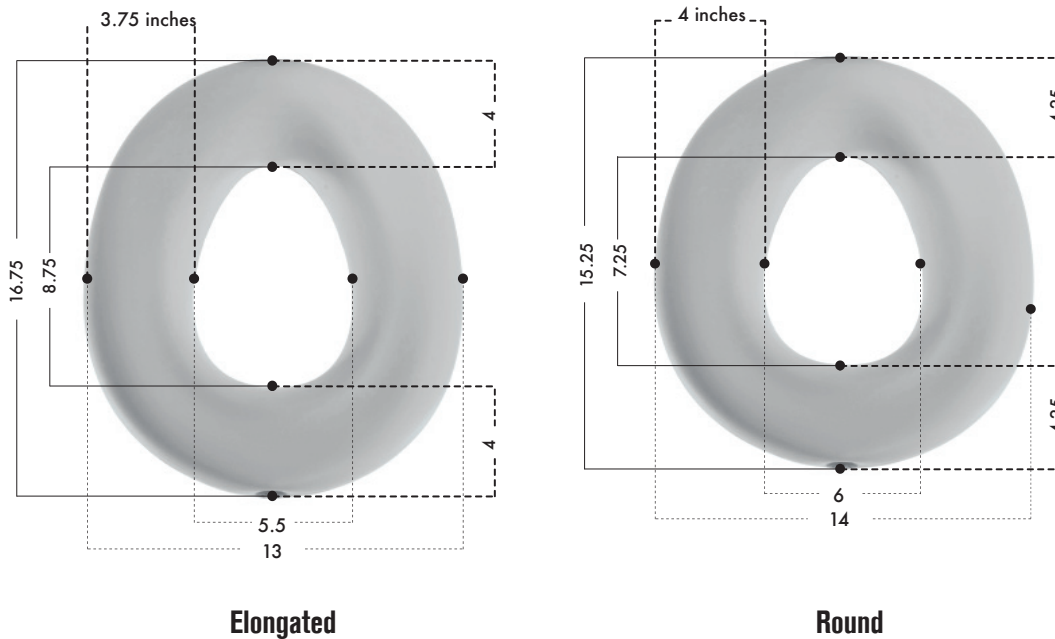

Limited Warranty

Bergeron by Design warrants to the original retail purchaser of the Special Tomato® product, that if any part or component proves defective in material or workmanship within five years of the purchase date, the defective part will be repaired or replaced (at Bergeron by Design's discretion) free of charge. Warranty service may be performed by an authorized service center or (at Bergeron by Design's discretion) the factory.

This warranty does not cover normal wear and tear, or damage caused by accident or misuse.

To exercise this limited warranty, the user should first obtain a Return Authorization Number from the dealer the Special Tomato® product was purchased from. The product must be delivered charges pre-paid (UPS recommended) to the factory or to an authorized service center, together with a copy of the original invoice, the Return Authorization Number and a written description of the problem.

Product Care

Special Tomato® Products are impermeable to fluids. Wipe with a mild detergent to clean periodically.

Do not submerge to prevent water from penetrating the air vent.

Customer Service

Contact your dealer directly. Or...

Toll Free: 1.866.529.8407
International: 1.315.429.8407
Fax: 1.315.429.8862
Web: www.specialtomato.com

**AUTHORISED EUROPEAN
REPRESENTATIVE**
GLOBAL CAREHAB
ROSENVANGS ALLE 32
8260 VIBY J
DENMARK
+45 24 65 55 50



Notice

No part of this document may be photocopied, reproduced, transmitted, transcribed, stored in a retrieval system or translated to another language or computer language, in any form or by any means, electronic, mechanical, magnetic, optical, chemical, manual, or otherwise without the prior written consent of Bergeron by Design.

Use only Special Tomato® accessories and parts on Special Tomato® products. Special Tomato® parts are not interchangeable with other manufacturers' products. Replace any worn parts immediately for safety. © 2013 Bergeron By Design.